


512 REASONS WHY IT'LL NEVER YOU LET YOU DOWN

Foxseal Vented is the first ever chest seal with 512 vents¹ giving you more ways to release pressure, prevent blockages and save a life.

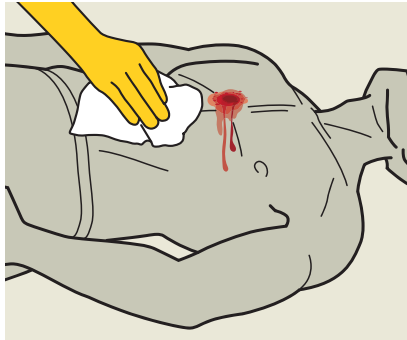

1. Wipe dirt/fluid from skin around the wound.



2. Pack has 4 tear notches, one at each corner. Tear open pack and remove Foxseal vented.




3. Grip tab and peel to remove Foxseal vented from clear liner to expose the adhesive side




4. Place dressing centred on the wound, adhesive side down.




5. Firmly press dressing to skin around wound to seal.

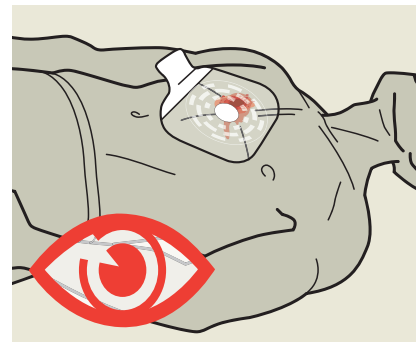

6. Continue to assess casualty and monitor action of chest seal.

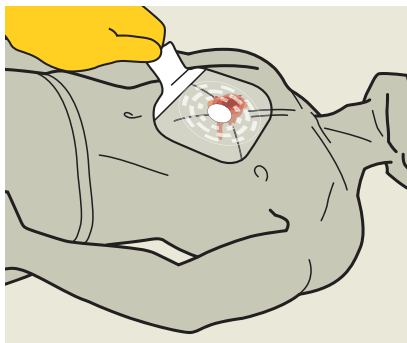

7. Use tab to vent and re-seal if required.

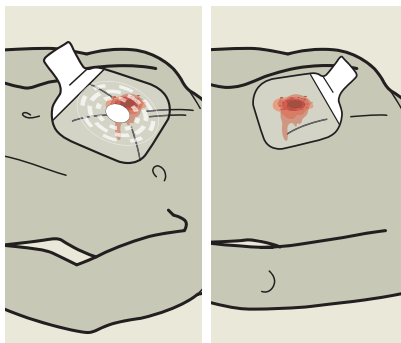

8. Use an additional dressing for each additional chest wound. Additional dressing does not need to be vented, unless that is the only dressing available. A manual occlusive dressing will suffice.

CELOX™ MEDICAL
MADE TO SAVE LIVES

Celox Medical
T: +44 (0)1270 500019
F: +44 (0)1270 500045
E: celoxenquiries@medtrade.co.uk
www.celoxmedical.com

For full details, instructions and warnings refer to manufacturer's instructions for use

1. A hydrogel based dressing with film roll stock and an additional Polyurethane (PU) film layer covering a central soft valve. Smart radial design consists of 3 circles, each with 8 vents. A total of 24 vents and 512 venting pathways that minimise the risk of occlusion and provide a secure one way venting system.
Copyright © 2020 Medtrade Products Ltd. CELOX™ and FOXSEAL™ is a trademark of Medtrade Products Ltd. All rights reserved.