


Product Range

Patient Moving & Handling

Reducing the risk of MSK injury
and pressure-related sores



Banana

Part of the GBUK Group 

GBUK Banana is one of Europe's leaders in the technology of patient moving and handling.

The Banana range includes lateral transfer boards and sheets used for moving patients from a bed to a trolley or operating table, slide sheets to assist in repositioning patients in bed, and hoists and slings for moving patients in safety and with dignity. Innovative and providing excellent value, GBUK Banana products directly enhance & improve the safety and well-being of patients and carers.

Think GBUK BANANA for:

Quality
Innovative
Clinician Led Solutions



GBUK Group award-winning specialist companies are expert in the provision of enteral feeding, critical care and patient moving and handling products and technologies. GBUK Group companies are suppliers to every UK NHS Trust and to a rapidly expanding Export customer base.

Within the Group, GBUK Enteral is the UK's leading supplier of enteral feeding tubes, syringes and gastrostomy devices, GBUK Healthcare offers single-use disposable medical, surgical & critical care devices, and GBUK Banana provides a comprehensive range of innovative safety centred patient moving and handling systems.


The Group's success is founded upon the pursuit of technical innovation and the provision of world class products in terms of both quality and value.

Product Range

Range Overview

Patient Repositioning, Proning & Transfer Devices

Single Patient Use Slide Sheet, Washable Slide Sheet, Disposable Slide Sheet, Versal, Intoprone, Orbit, Transfer Sheet, Lateral Folding, Lateral Rigid, Rolaslide, Prime Seated, Q2 Seated and Original Seated

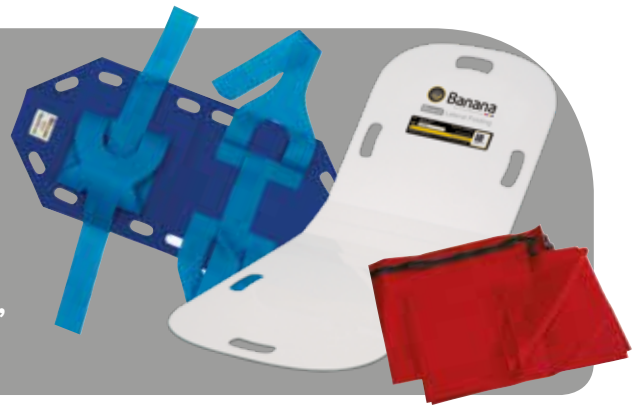

Patient Elevation

Pro-Lock Sling
and Pro-Loop Sling



Air Assisted Devices

Air Assisted Transfer Platform,
Air Single Patient Transfer and Airseat



Care Management Systems & Accessories

Versal On Bed System, Versal System Adaptors,
Versal System Clamp, One Way Glide, Turning,
Transfer Handles and Belt



SPU Sheet

Strong and durable with high slip performance

a strong and durable red hemmed slide sheet. Silicone coated on both sides with a low profile seam, this sheet provides long lasting high slip characteristics to enable safe and easy patient repositioning and transfers.

Strong hemmed sheet
optimum durability

**Silicone coated
with high slip finish**
safe and easy
patient movement

**Think smart,
think safe...**

When moving
a patient
**always use
slide sheets**



Patient Repositioning

- **Hanging strap**
easy storage close to the patient

- Available in a wide choice of sizes

- **Latex free**
minimises risk of allergic reaction

Repositioning Devices from GBUK Banana offer

- ✓ Innovative products that have been developed by expert clinicians
- ✓ Advanced materials and design technology
- ✓ Scope for full compliance with manual handling techniques
- ✓ Potential to reduce the risk of injury and increase efficiency & standards in practice



Banana Slide

Single Patient Flat & Tubular

Slide Sheet SPU

- ✓ Low coefficient of friction, coated both sides for reduced forces required to move a patient & ensuring equal abilities both sides
- ✓ Large range of flat sheet without handles, flat sheets with handles, tubular slide sheets and the Versal sheet for a one size sheet that fits all procedures for repositioning, transfer and turning
- ✓ Hemmed or unhemmed options
- ✓ Soft 60gsm fabric in red or yellow for patient comfort
- ✓ All sheets are Latex free and individually packed

SPU Sheet

Soft and quiet in use for maximum patient comfort

an unhemmed yellow slide sheet with a specially formulated silicone coating. Soft to the touch & quiet in use with excellent slip characteristics. This sheet provides an ideal choice for comfortable, stress free patient transfer and repositioning.



● **Strong unhemmed sheet**
optimum durability

● **Silicone coated with high slip finish**
safe and easy patient movement

Patient Repositioning

Hanging strap

easy storage close to the patient

Repositioning Devices from GBUK Banana offer

- ✓ Innovative products that have been developed by expert clinicians
- ✓ Advanced materials and design technology
- ✓ Scope for full compliance with manual handling techniques
- ✓ Potential to reduce the risk of injury and increase efficiency & standards in practice



Banana Slide

Single Patient Flat & Tubular

Slide Sheet SPU

- ✓ Low coefficient of friction, coated both sides for reduced forces required to move a patient & ensuring equal abilities both sides
- ✓ Large range of flat sheet without handles, flat sheets with handles, tubular slide sheets and the Versal sheet for a one size sheet that fits all procedures for repositioning, transfer and turning
- ✓ Hemmed or unhemmed options
- ✓ Soft 60gsm fabric in red or yellow for patient comfort
- ✓ All sheets are Latex free and individually packed

Latex free

minimises risk of allergic reaction

Available in a wide choice of sizes

Washable

4 times longer effective slip than competitor sheets*

a remarkably durable hemmed slide sheet. Proven in wash trials to maintain effective slip characteristics 4 times longer than competitor sheets.*

With a specially designed silicone coating, this sheet is soft to the touch and quiet in use. An ideal choice for comfortable, stress free patient transfer and repositioning.

Maintains effective slip 4 times longer than competitors
cost effective

Latex free –
minimises risk of allergic reaction

Patient Repositioning

Unique silicone coating

soft to the touch and
quiet in use

Individually packed

reduces unnecessary
laundering, risk of cross
contamination

Repositioning Devices from GBUK Banana offers

- ✓ Innovative products that have been developed by expert clinicians
- ✓ Advanced materials and design technology
- ✓ Scope for full compliance with manual handling techniques
- ✓ Potential to reduce the risk of injury and increase efficiency & standards in practice



Banana Slide

Single Patient Flat & Tubular

Slide Sheet Washable

- ✓ Hemmed, low coefficient of friction sheets, coated both sides for reduced forces required to move a patient and ensuring equal abilities both sides
- ✓ Large range of flat sheet without handles, flat sheets with handles, tubular slide sheets and the Versal sheet for a one size sheet that fits all procedures for repositioning, transfer and turning
- ✓ Soft 71gsm fabric for improved patient comfort in teal. Proven in independent wash trials to have lasted 4 times longer lasting effective slip performance than leading competitors and extremely durable
- ✓ All sheets are latex free and individually packed


Disposable

Perforated roll format for convenient low cost use

a strong triple layer waterproof sheet which is designed to maintain its integrity when wet and dry.

With excellent slip characteristics and delivered in a convenient perforated roll format, this sheet is the ideal choice where an easy to use, low cost option is required.

Available in a choice of sizes

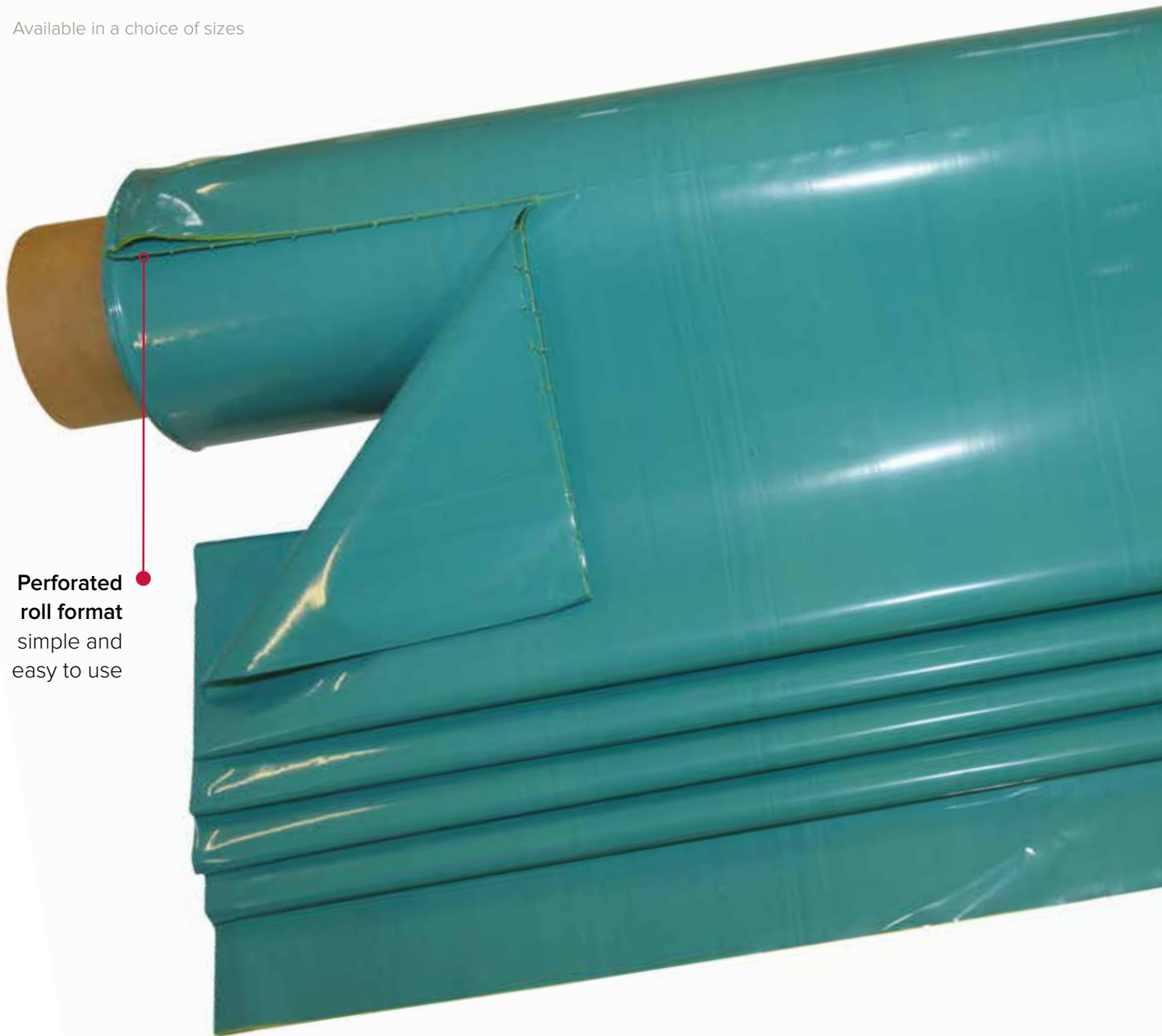

**Perforated
roll format**
simple and
easy to use

Patient Repositioning

Waterproof triple layer polymer
high strength and slip characteristics

Latex free
minimises the risk
of allergic reaction

Repositioning Devices from GBUK Banana offer

- ✓ Innovative products that have been developed by expert clinicians
- ✓ Advanced materials and design technology
- ✓ Scope for full compliance with manual handling techniques
- ✓ Potential to reduce the risk of injury and increase efficiency & standards in practice



Banana Slide

Single Patient Flat & Tubular

Slide Sheet Disposable

- ✓ Improved material to ensure patient comfort with a strong triple layer waterproof sheet which is designed to maintain its integrity when wet & dry
- ✓ With excellent slip characteristics and delivered in a convenient perforated roll format, this sheet is the ideal choice when an easy to use, lowest cost option is required
- ✓ All sheets are Latex free
- ✓ Available in two forms of tubular and one standard flat sheet option

Versal

System

Versal System has been designed by Diane Hindson, Back care Advisor from County Durham and Darlington NHS Foundation Trust, the new Banana Versal developed by GBUK Healthcare is a moving and handling slide sheet with a difference.

Versal replaces the need for multiple flat or tubular slide sheets. The unique design facilitates a full range of patient movement, with the versatility of both flat and tubular sheets.

Versal is a single sized sheet, placed under the patient in the same position, for every procedure.



Facilitates medial and lateral movements

Banana Versal the next generation slide sheets are available in

Washable

Single Patient Use

Breathable SPU



Patient Repositioning



One size for all Manual Handling movements

Banana Versal

Helps reduce the risk of injury

Helps avoid hazardous manual handling operations

Helps make proper use of equipment provided for safety

Helps ensure that activities do not put others at risk

Versal System from GBUK Banana

- ✓ Simplifies and helps reduce confusion of slide sheet selection by having one sheet for all repositions medially and laterally
- ✓ Improve patient safety by ensuring all bony prominences of the patient are protected by two layers of slide sheet
(As researched by Loughborough University)
- ✓ Can improve patient comfort, reduce nursing time and improve efficiency of care
- ✓ Can help reduce the hidden costs of pressure ulcers or an MSK injury

Banana Versal Evaluation Results

Respondents who rated Banana Versal as good or excellent:

Using one Versal for all procedures	94%
Using just one Versal for turning	100%
Using one Versal for repositioning	94%
Using one Versal for transferring	86%
Using one Versal for proning	86%
Considering the feel of the Versal sheet	97%
To remain under ventilated patients	87%
Respondents who would be happy to use Banana Versal	94%

Date on file 2016, GBUK Ltd

Versal helps reduce user error & risk, improves patient dignity, comfort and safety, and helps protect staff from injury.

HIGHLY COMMENDED
Nurse Awards 2016



PATIENT SAFETY AWARDS FINALIST 2017

Prone

Intoprone & Orbit

Unique easy to use one piece proning device

Banana Intoprone is a unique, easy to use one piece device for accurately turning patients into a prone position. Staff maintain complete control during the entire turn and no longer have to “catch” the patient which reduces the risk of staff injury and improves patient safety.

Eliminates
need to “catch”
the patient
reduces risk
of staff injury

One piece device
easy to insert
and remove



Complete control
of patient during
proning improves
patient safety

Watch the instructional video at

www.gbukbanana.com

Patient Proning



Proning Devices from GBUK Banana offer

- ✓ Safe solutions to patient proning
- ✓ Reduced risk to staff and patient
- ✓ Time saving for clinicians
- ✓ A more comfortable & dignified turn for the patient
- ✓ Fully-controlled patient proning and positioning

87% of participants concluded that the overall safety and ease of use of Banana Intoprone was

“Very good or excellent” in a large acute Trust evaluation.*

Flexible

suitable for use with a range of proning supports

Individually packed reduces risk of cross contamination or wastage

Banana Intoprone – Proning device

- ✓ Complete control of patient during proning – improves patient safety
- ✓ Eliminates need to “catch” the patient – reduces risk of staff injury
- ✓ One piece device – easy to insert and remove
- ✓ Individually packed – reduces risk of cross contamination or wastage
- ✓ Flexible – suitable for use with a range of proning supports

Banana Orbit – Patient proning devices this time with a difference

- ✓ No more need for ‘Flip & catch’
- ✓ Simple two piece device which safely prones the patient and then returns them into supine position in a fully controlled procedure

*Results from an evaluation on Banana Intoprone positioning device at Bradford Royal Infirmary. Data on File July 2014

Transfer Sheet

Unique handle design for controlled patient transfer

manufactured from a specially designed polypropylene non-woven fabric. This transfer sheet has a material strength of 500kg when wet and dry.

High lateral slip characteristics
stress free patient transfer

**High material
strength (500kg SWL)
when wet or dry**
suitable for all patients



Patient Transfer

Individually packed

reduces risk of cross contamination

Transfer Devices from GBUK Banana offer

- ✓ A wide range of innovative products
- ✓ Compliance with safe manual handling procedures
- ✓ Increased safety for carer & the patient

With six ergonomically positioned handles

Which ensure that the patient's weight is evenly distributed and high lateral slip characteristics, Banana Transfer provides smooth, stress free patient transfers for carers.

Six ergonomically positioned handles

even distribution
of patient's weight
during transfer



**Designed for use with
Banana Transfer Handles**
reduces risk of injury



Lateral Folding

Unique folding mechanism for semi upright patient transfer. Made from high strength polypropylene. The unique design of this hinged board means that it can be folded to allow the transfer of patients in a semi upright as well as a supine position. This is beneficial for patients who suffer from breathing difficulties or who find it difficult to lie flat due to their underlying medical conditions.

High strength polypropylene
for maximum durability

Easy to wipe clean



Patient Transfer



Transfer Devices from GBUK Banana offer

- ✓ A wide range of innovative products
- ✓ Compliance with safe manual handling procedures
- ✓ Increased safety for carer and the patient

Lightweight with six ergonomically

Positioned handles, this board can be easily positioned and removed in all transfer situations and requires minimal storage space. Two lengths and two widths of board are available, the narrower 59cm board is ideal for use in confined areas, whereas the wider board will effectively bridge a gap of up to 18cm.

Unique handle design for controlled patient transfer

- ✓ Smooth, low friction surface for safe patient transfer
- ✓ High strength polypropylene for maximum durability
- ✓ Lightweight with six handholds for easy positioning and removal
- ✓ Anti slip pads decrease risk of unwanted board movement in transfer
- ✓ Choice of widths and lengths to meet user needs
- ✓ Easy to clean

Lateral Rigid

Smooth low friction surface for stress free patient transfer


Made from high strength polypropylene. Banana Board – Lateral Rigid features an exceptionally smooth, low friction surface for the safe transfer of patients in a supine condition.

With six integral handhold's, a lightweight profile and curved ends, this board can be easily positioned and removed in all transfer situations. Anti slip pads minimise the risk of unwanted board movement during transfer so improving patient safety.

Two lengths and two widths of board are available to meet user needs and boards can be wiped clean using soap and water, cleaning disinfectants or alcohol based solutions (up to 75%).

High strength
polypropylene
for maximum
durability

Smooth, low friction
surface for safe
patient transfer

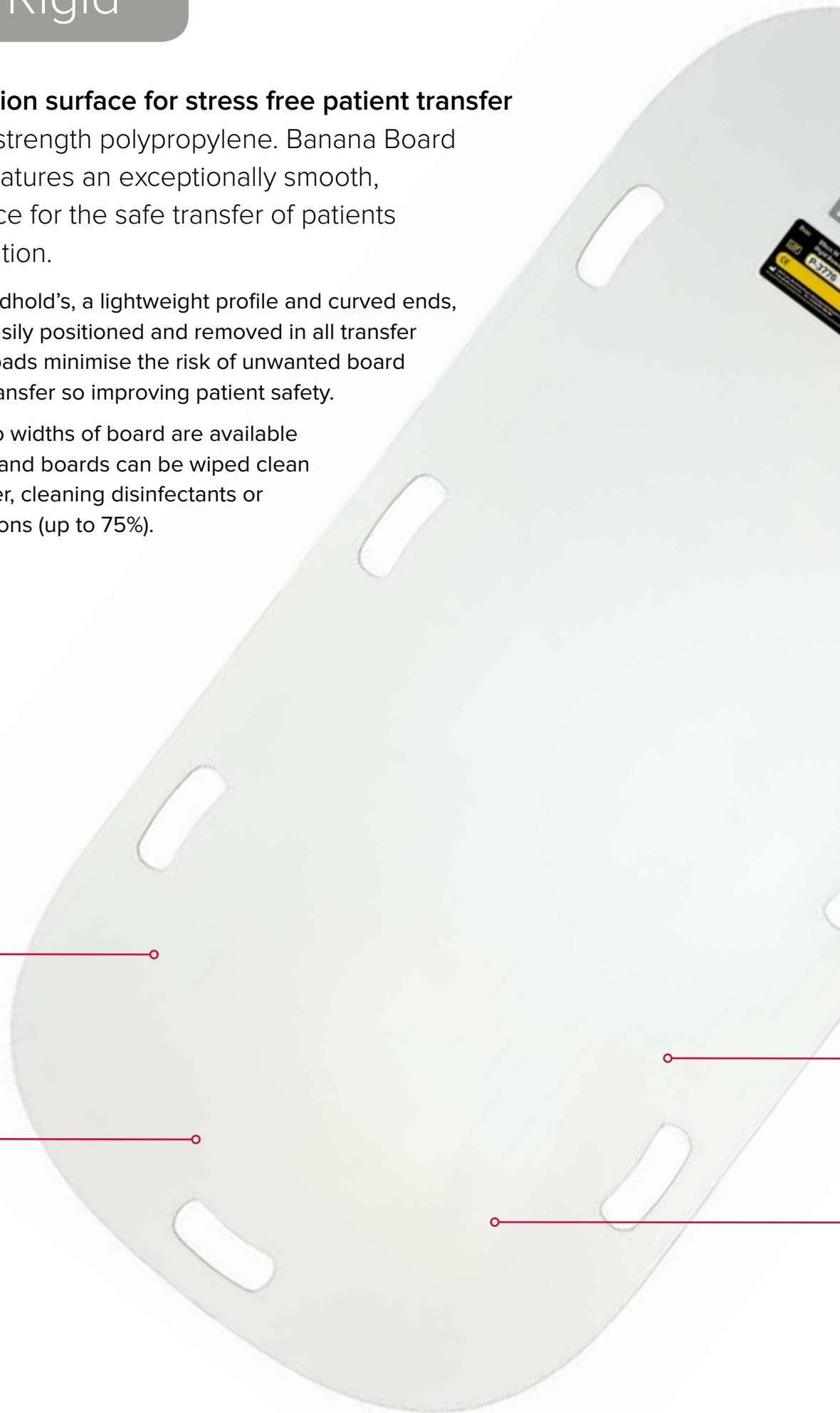

Patient Transfer



Easy to clean

Transfer Devices from GBUK Banana offer

- ✓ A wide range of innovative products
- ✓ Compliance with safe manual handling procedures
- ✓ Increased safety for carer and the patient

Lightweight with six ergonomically

Positioned handles, this board can be easily positioned and removed in all transfer situations and requires minimal storage space. Two lengths and two widths of board are available, the narrower 59cm board is ideal for use in confined areas, whereas the wider board will effectively bridge a gap of up to 18cm.

Unique handle design for controlled patient transfer

- ✓ Smooth, low friction surface for safe patient transfer
- ✓ High strength polypropylene for maximum durability
- ✓ Lightweight with six handholds for easy positioning and removal
- ✓ Anti slip pads decrease risk of unwanted board movement in transfer
- ✓ Choice of widths and lengths to meet user needs
- ✓ Easy to clean

Choice of widths and lengths to meet user needs

Lightweight with six handholds for easy positioning and removal

Anti slip pads decrease risk of unwanted board movement in transfer

Board

Rolaslide

Rolaslide is a lightweight foldable patient transfer system comprising a rolling slide board, 2 Banana Transfer sheets and a pack of 10 disposable roll board sleeves. This system enables patients to be transferred safely and comfortably with minimal effort or risk of injury to carers.

Lightweight rolling slide board
for safe, comfortable transfers
by a single carer



Patient Transfer



Foldable
minimises storage
space required

Rollable sleeve
protects board from
contamination

Transfer Devices from GBUK Banana offer

- ✓ A wide range of innovative products
- ✓ Compliance with safe manual handling procedures
- ✓ Increased safety for carer and the patient

Rolaslide features

A cushioned board for extra patient comfort during transfer. The lightweight design makes Rolaslide highly manoeuvrable and easy to work with. Rolaslide can be folded in half for convenient transport and compact storage. A disposable sleeve protects the Rolaslide from accidental soiling.



Board

Prime Seated

The Banana Board PRIME Seated is designed to assist in the seated transfer of a patient between two surfaces. Transfers from bed to wheelchair, wheelchair to chair, wheelchair to toilet etc are just some of the uses of this board. The Banana Board PRIME Seated is 620mm wide x 260mm deep x 12/6mm thick & weighs 1kg.


Lightweight, easy to hold and carry

Low cost hard-wearing board

Banana Board
Part of the OBUS Group
Prime Seated
200kg
Ref Order Code: BB-PRIME
www.banana.com +44 (0)1753 288 587

200kg weight limit (30st)

Tapered edges to ensure easy insert and removal

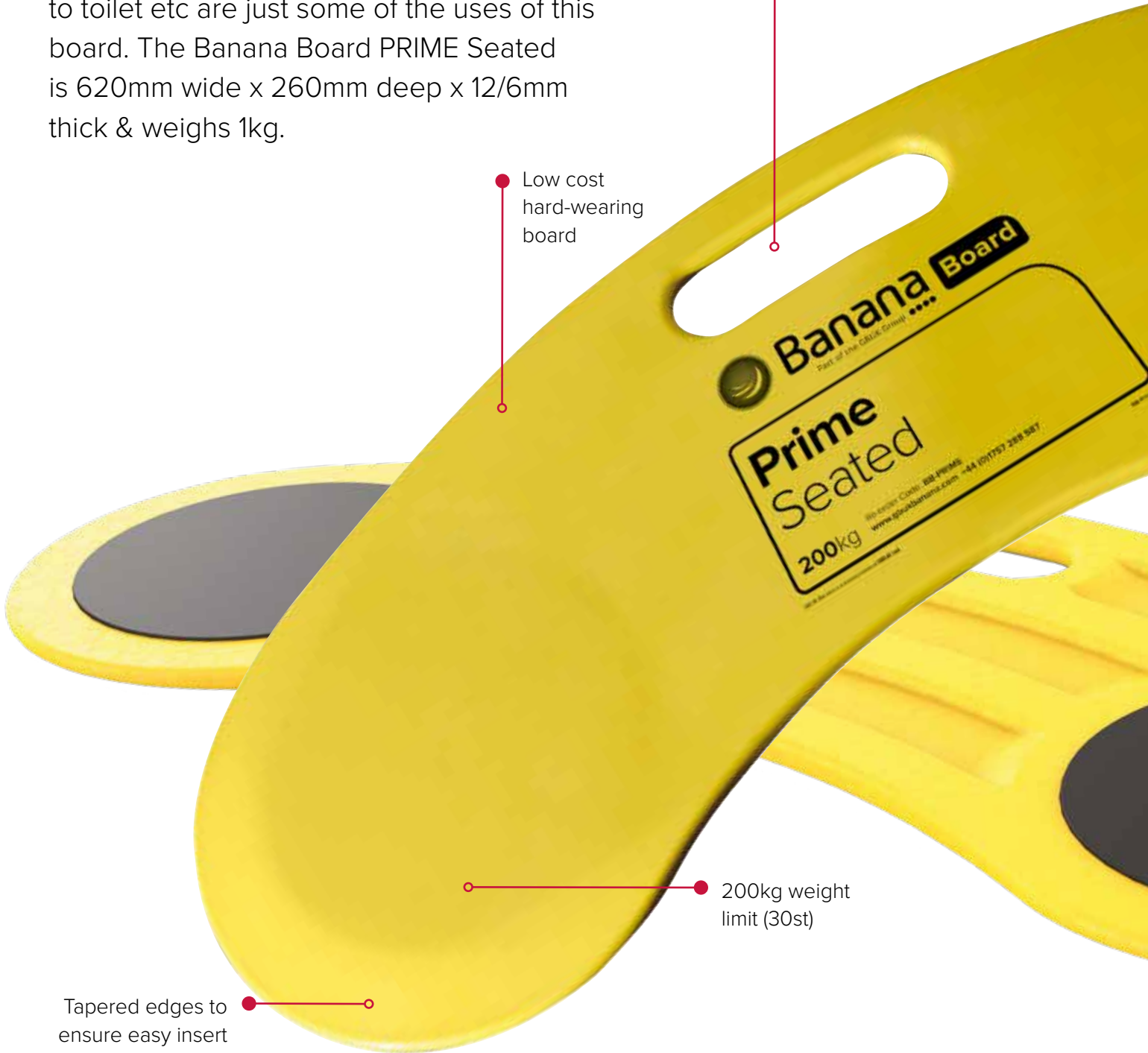

Patient Transfer

Transfer Devices from GBUK Banana offer

- ✓ A wide range of innovative products
- ✓ Compliance with safe manual handling procedures
- ✓ Increased safety for carer & the patient

The new Banana Board PRIME incorporates all the features of the patented Q2 Board, with the additional advantage that it is slightly smaller than its predecessor.

The Banana Board PRIME has an impressive market-leading maximum weight capacity of 200kg making it ideal for all general and paediatric use. The Banana PRIME is the choice for a quality, comfortable, lightweight seated transfer board!



Curved board shape allows side-to-side and straight transfers

Supports independent movement

Non Slip Pads to ensure safe movement

The Banana PRIME is the choice for a quality, comfortable, lightweight seated transfer board!

Q2 Seated Transfer Board With an exceptionally high and independently tested safe working load limit, the patented universal design of this seated transfer board makes it ideal for use by children, adults and bariatric patients.



Transfer Devices from GBUK Banana offer

- ✓ A wide range of innovative products
- ✓ Compliance with safe manual handling procedures
- ✓ Increased safety for carer and the patient

Board

Original Seated

Original Banana Transfer Board

The authentic patented Banana Board widely used by a broad range of patients for the last 25 years.

Curved board shape allows side to side transfers

Non slip pads to ensure safe movement

Safety tested SWL of 145kg (23 Stone)

Transfer Devices from GBUK Banana offer

- ✓ A wide range of innovative products
- ✓ Compliance with safe manual handling procedures
- ✓ Increased safety for carer and the patient

Board

Straight Seated

Straight Seated Board

has a straight profile of this board has been designed for easy use in confined areas where only a straight transfer is possible.

Anti slip pads for maximum grip during transfers

Straight board shape facilitates transfer in confined areas

Safety tested SWL of 175kg (27 Stone)

Transfer Devices from GBUK Banana offer

- ✓ A wide range of innovative products
- ✓ Compliance with safe manual handling procedures
- ✓ Increased safety for carer and the patient

Slings

Lock Sling

Patient Elevation

Pro-Lock® Disposable Sling, has been developed to provide a cost effective patient handling solution to the increasing challenges of infection control. Available in four sizes (x-small, small, medium and large), the Pro-Lock® is designed to be issued for single patient use only and to be disposed of when soiled or no longer required.


The Pro-Lock® Sling incorporates a head support for increased comfort and a Patented Positive Locking Clip fixing, suitable for use with any hoist with a clip fixing mechanism.

Positive Locking Clip Fixing
- Patent Applied For: GB0821240.9



230kg
weight limit

GBUK has a wide range of slings for Clip and Loop hoists to help perform every procedure, comfortably and safely. The clip slings have a patented design to ensure safety when lifting a patient.

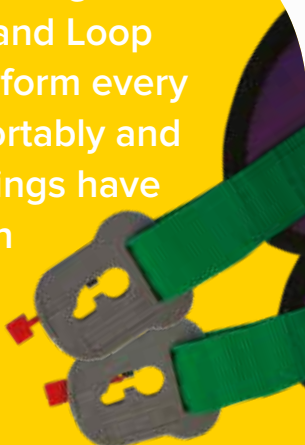

Slings - Pro-Lock Sling

- ✓ Infection control maximised
- ✓ Laundry costs eliminated
- ✓ Unique safety harness configuration
- ✓ 230kg weight limit

Slings

Loop Sling

Patient Elevation

Pro-Lock® Disposable Sling, Have been developed to provide a cost effective patient handling solution to the increasing challenges of infection control. Available in three sizes (small, medium, large), the Pro-Loop is designed to be issued for single patient use only and to be disposed of when soiled or no longer required.

The Pro-Loop incorporates a head support for increased comfort, extra padded legs, a unique safety harness configuration and is suitable for use with any hoist with a standard loop attachment spreader bar.




Patient Elevation from GBUK Banana offer

- ✓ The complete package for assisting patients who have fallen
- ✓ A solution for moving patients from a supine to a seated position
- ✓ A simplified solution for patient elevation in all circumstances
- ✓ An enhanced level of patient comfort & safety

Slings - Pro-Loop

- ✓ Infection control maximised
- ✓ Laundry costs eliminated.
- ✓ Unique safety harness configuration
- ✓ 230kg weight limit

GBUK has a wide range of slings for Clip and Loop hoists to help perform every procedure, comfortably and safely. The clip slings have a patented design to ensure safety when lifting a patient.

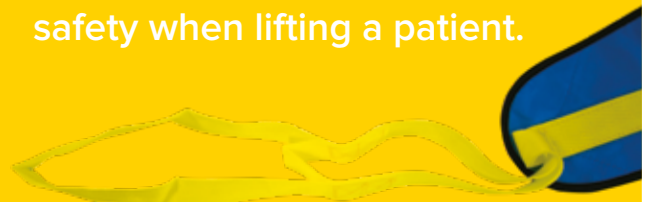

Air

Reusable

Air Assisted Transfer Platform the uniquely designed Air platform provides a safe, comfortable and dignified method of transferring and positioning patients. It also enables caregivers to perform lateral transfers with relative ease and with minimal risk of injury.



Patient Elevation



Air Assisted Devices from GBUK Banana offer

- ✓ A superior level of comfort and safety for carers and patients
- ✓ A range of innovative and easy-to-use appliances
- ✓ Scope for fewer carers to be needed to carry out a patient movement
- ✓ A superior level of patient dignity during movement and handling procedures

Made from specialised medical fabrics, this cushioned Air platform maintains patient comfort whilst in situ and minimises friction and shear forces during transfer. Its patented extended handles enable staff to maintain a proper posture protecting them from the risk of injury.

Air Assisted Transfer Platform

- ✓ Cushioned platform
- ✓ Specialised fabrics
- ✓ Stabilisation bars
- ✓ Long extension handles
- ✓ Protective Sani-Liner
- ✓ Colour coding scheme
- ✓ Safe working limit of 450kg

Air

Single Patient Use

Single Patient Transfer system features a rugged design that is perfect for multiple lateral transfers with the same patient. Designed to optimise patient comfort during transfers, enhance infection control procedures and protect staff from injury, the Air SPS is suitable for use with all patients.

Cushioned
platform

Specialised
fabrics

Long extension
handles



Patient Elevation

Air Assisted Devices from GBUK Banana offer

- ✓ A superior level of comfort and safety for carers and patients
- ✓ A range of innovative and easy-to-use appliances
- ✓ Scope for fewer carers to be needed to carry out a patient movement
- ✓ A superior level of patient dignity during movement and handling procedures

Colour coding scheme

Safe working limit of 450kg



Versal

On Bed System

Accessories

Versal On Bed System a tough, durable adaptor frame that can be easily fitted to profiling beds to provide attachment points for the Banana Versal Sheet Clamp.

The Banana Versal Bed Adaptor can be fitted to most types of beds including electric profiling, static frame and divan bases in both acute and community care environments.

It is designed to be used in conjunction with the Banana Versal Sheet Clamp to secure slide sheets or in-bed repositioning systems to prevent unwanted movement once the occupant of the bed is in a comfortable position.



Air

Airseat

Accessories

The Banana Air Seat

is a foldable and inflatable seat cushion which provides excellent postural support to the pelvic area and may also be used as an alternative to lumbar support products. It can help to provide some comfort to those who may suffer pain when sitting because of injury, joint pain or bony prominences, whilst also blocking cold air from the floor.



Versal

System Adaptor

Versal System Adaptor

can be fitted to most types of beds including electric profiling, static frame and divan bases in both acute and community care environments.

Can be fitted in minutes to most types of bed without the need for any modifications.



Polypropylene construction ensures the Banana Versal Bed Adaptor can be easily cleaned in compliance with local infection control policies.

Accessories

Versal


System Clamp

Versal System Clamp

is a versatile Sheet Clamp used in conjunction with the Banana Versal Bed Adaptor to prevent unwanted movement of in-situ repositioning sheet systems.

The Banana Versal Sheet Clamp can be fitted to a suitable slide sheet or in-bed repositioning system to enable a hoist assisted turn.

Each Banana Versal Sheet Clamp is supplied with a Banana Versal Clamp Strap which when fitted to the Banana Versal Sheet Clamp can be attached to hook onto a hoist spreader bar.

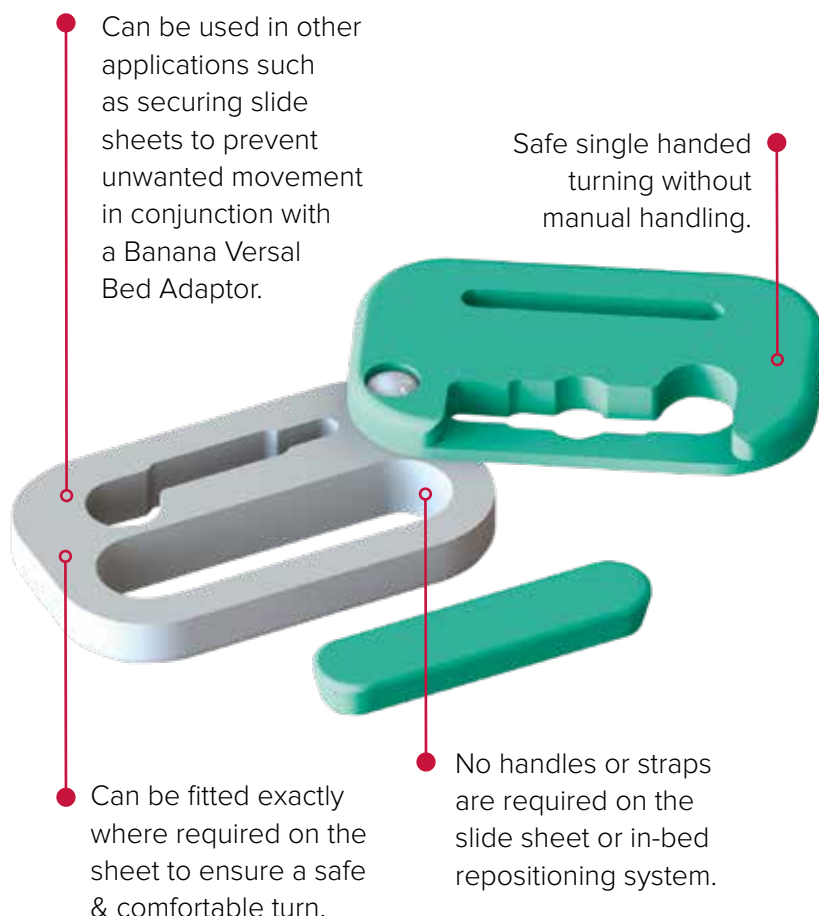

Can be used in other applications such as securing slide sheets to prevent unwanted movement in conjunction with a Banana Versal Bed Adaptor.

Safe single handed turning without manual handling.

Can be fitted exactly where required on the sheet to ensure a safe & comfortable turn.

No handles or straps are required on the slide sheet or in-bed repositioning system.

Accessories

Accessories

One Way Glide

One Way Glide allows movement in one direction only to prevent patients continually slipping forward. Posture is improved, and patients can remain sitting comfortably and safely in a chair.



Cushioned
for maximum
patient comfort

Ideal size for
most chairs
(38x48cm)

Accessories

Accessories

Turning

Turning is a strong and lightweight plastic turntable which provides easy rotation through 360 degrees for standing and seated transfers. With a diameter of 40.5cm and weighing just 1.35kg, it supports a maximum user weight of 115kg (18 stone).




Accessories

Accessories

Transfer Handle

Transfer Handles are made from high strength polypropylene, Banana Transfer Handles are designed to eliminate the need for staff to overreach during lateral transfers and reduce the risk of injury.

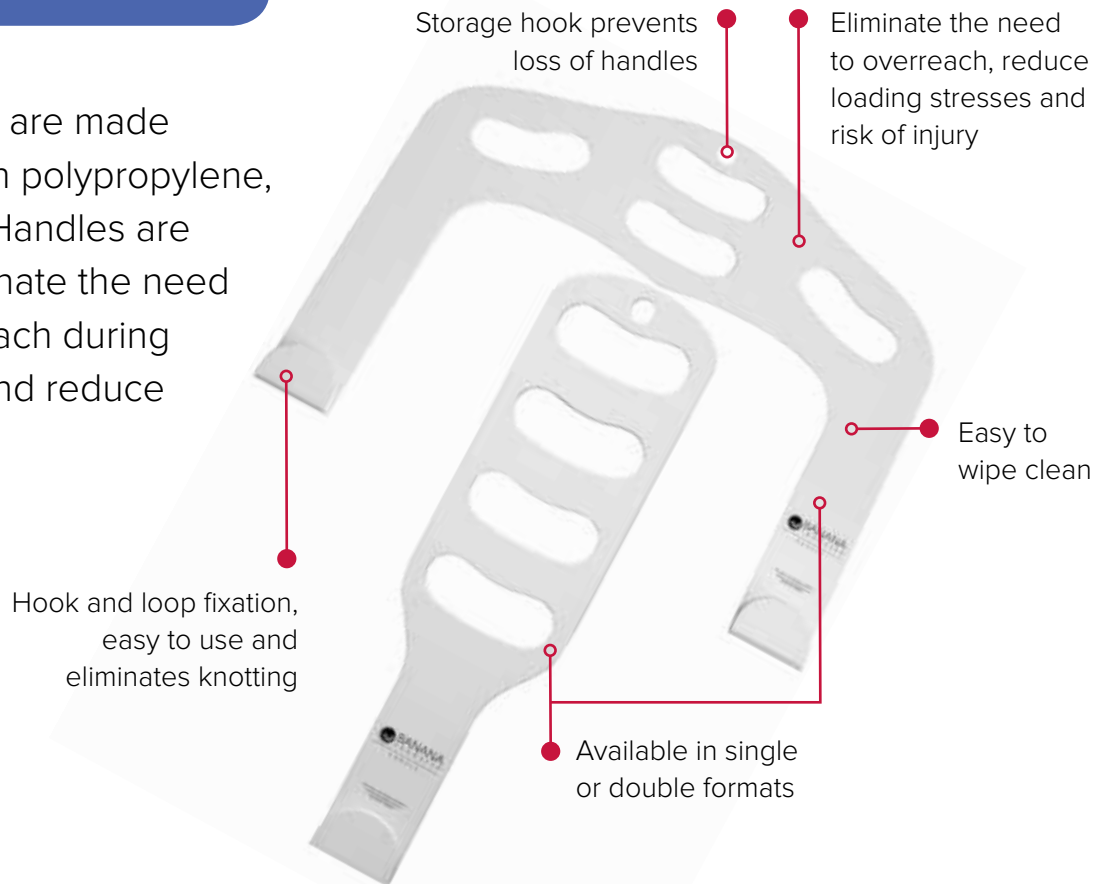

Accessories

Handling Belt

Handling Belt is designed to provide greater control for carers supporting patients with transfers and helps prevent the need to grab hold of sensitive limbs.



Accessories

Patient Repositioning, Proning & Transfer Devices Product Codes

	Product Code	Product Description	UOI	NHS SC Code
Slide SPU Sheet	DGF072200	Slide, Flat Slide Sheet Unhemmed, Single Patient Use, 72cm x 200cm	10 & 50	
	DGF072200H	Slide, Flat Slide Sheet with Handles Unhemmed, Single Patient Use, 72cm x 200cm	10 & 50	
	DGF100200	Slide, Flat Slide Sheet Unhemmed, Single Patient Use, 100cm x 200cm	10 & 50	
	DGF100200H	Slide, Flat Slide Sheet with Handles Unhemmed, Single Patient Use, 100cm x 200cm	10 & 50	
	DGF140200	Slide, Flat Slide Sheet Unhemmed, Single Patient Use, 140cm x 200cm	10 & 50	
	DGF140200H	Slide, Flat Slide Sheet with Handles Unhemmed, Single Patient Use, 140cm x 200cm	10 & 50	
	DGT072072	Slide, Tubular Slide Sheet Unhemmed, Single Patient Use, 72cm x 72cm	10. / 50	
	DGT072120	Slide, Tubular Slide Sheet Unhemmed, Single Patient Use, 72cm x 120cm	10 & 50	
	DGT072150	Slide, Tubular Slide Sheet Unhemmed, Single Patient Use, 72cm x 150cm	10 & 50	
	DGT072200	Slide, Tubular Slide Sheet Unhemmed, Single Patient Use, 72cm x 200cm	10 & 50	
	DGT100100	Slide, Tubular Slide Sheet Unhemmed, Single Patient Use, 100cm x 100cm	10 & 50	
	DGT100200	Slide, Tubular Slide Sheet Unhemmed, Single Patient Use, 100cm x 200cm	10 & 50	
	DGT200070	Slide, Tubular Slide Sheet Unhemmed, Single Patient Use, 200cm x 70cm	10 & 50	
	DGT200140	Slide, Tubular Slide Sheet Unhemmed, Single Patient Use, 200cm x 140cm	10 & 50	
	EGF072200	Slide, Flat Slide Sheet Hemmed, Single Patient Use, 72cm x 200cm	10 & 50	
	EGF072200H	Slide, Flat Slide Sheet with Handles Hemmed, Single Patient Use, 72cm x 200cm	10 & 50	
	EGF090200	Slide, Flat Slide Sheet Hemmed, Single Patient Use, 90cm x 200cm	10 & 50	
	EGF090200H	Slide, Flat Slide Sheet with Handles Hemmed, Single Patient Use, 90cm x 200cm	10 & 50	
	EGF140200	Slide, Flat Slide Sheet Hemmed, Single Patient Use, 140cm x 200cm	10 & 50	
	EGF140200H	Slide, Flat Slide Sheet with Handles Hemmed, Single Patient Use, 140cm x 200cm	10 & 50	
	EGT072072	Slide, Tubular Slide Sheet Hemmed, Single Patient Use, 72cm x 72cm	10 & 50	
	EGT072120	Slide, Tubular Slide Sheet Hemmed, Single Patient Use, 72cm x 120cm	10 & 50	
	EGT072150	Slide, Tubular Slide Sheet Hemmed, Single Patient Use, 72cm x 150cm	10 & 50	
	EGT072200	Slide, Tubular Slide Sheet Hemmed, Single Patient Use, 72cm x 200cm	10 & 50	
	EGT100100	Slide, Tubular Slide Sheet Hemmed, Single Patient Use, 100cm x 100cm	10 & 50	
	EGT100150	Slide, Tubular Slide Sheet Hemmed, Single Patient Use, 100cm x 150cm	10 & 50	
	EGT100200	Slide, Tubular Slide Sheet Hemmed, Single Patient Use, 100cm x 200cm	10 & 50	
	EGF100200	Slide, Flat Slide Sheet Hemmed, Single Patient Use, 100cm x 200cm	10 & 50	
	EGT140200	Slide, Tubular Slide Sheet Hemmed, Single Patient Use, 140cm x 200cm	10 & 50	
	EGF072200	Slide, Flat Slide Sheet Hemmed, Single Patient Use, 72cm x 200cm	10 & 50	
	EGF090200	Slide, Flat Slide Sheet Hemmed, Single Patient Use, 90cm x 200cm	10 & 50	
	EGF140200	Slide, Flat Slide Sheet Hemmed, Single Patient Use, 140cm x 200cm	10 & 50	
	EGT072072	Slide, Tubular Slide Sheet Hemmed, Single Patient Use, 72cm x 72cm	10 & 50	
	EGF100200	Slide, Flat Slide Sheet Hemmed, Single Patient Use, 100cm x 200cm	10 & 50	
	EGT140200	Slide, Tubular Slide Sheet Hemmed, Single Patient Use, 140cm x 200cm	10 & 50	
	B-EGF072200P	SPU Double Pack 2x EGF072200 Sealed together Placed in a drawstring bag, 72cm x 200cm	25	
	B-EGF100200P	SPU Double Pack 2x EGF100200 Sealed together Placed in a drawstring bag, 100cm x 200cm	25	
	B-EGF140200P	SPU Double Pack 2x EGF140200 Sealed together Placed in a drawstring bag, 140cm x 200cm	25	
	PATPAQ-EGF090200/H	SPU Pack Containing Individually Sealed 1x EGF090200 & 1x EGF090200H + Drawstring Bag, with Handles, 90cm x 200cm	25	
	PATPAQ-EGF100200	SPU Pack Containing Individually Sealed 1x EGF100200 Red & 1x EGF100200 Yellow + Drawstring Bag, 100cm x 200cm	25	
	PATPAQ-EGF100200/H	SPU Pack Containing Individually Sealed 1x EGF100200 Red & 1x EGF100200H Yellow + Drawstring Bag, with Handles, 100cm x 120cm	25	
	PATPAQ-EGF140200/H	SPU Pack Containing Individually Sealed 1x EGF140200 & 1x EGF140200H + Drawstring Bag, with Handles, 140cm x 120cm	25	
	PATPAQ-EGT100100	SPU Pack Containing Individually Sealed 2x EGT100100 + Drawstring Bag, 100cm x 100cm	10	

Product Codes

Product Range

Washable Slide Sheet Product Codes

	Product Code	Product Description	UOI	NHS SC Code
Slide Washable	LGF070150H	Slide, Flat Slide Sheet with Handles, Washable, 70cm x 150cm	10 & 50	
	LGF070200	Slide, Flat Slide Sheet, Washable, 70cm x 200cm	10 & 50	
	LGF070200H	Slide, Flat Slide Sheet with Handles, Washable, 70cm x 200cm	10 & 50	
	LGF100150H	Slide, Flat Slide Sheet with Handles, Washable, 100cm x 150cm	10 & 50	
	LGF100200	Slide, Flat Slide Sheet, Washable, 100cm x 200cm	10 & 50	
	LGF100200H	Slide, Flat Slide Sheet with Handles, Washable, 100cm x 200cm	10 & 50	
	LGF110200	Slide, Flat Slide Sheet, Washable, 110cm x 200cm	10 & 50	
	LGF140200	Slide, Flat Slide Sheet, Washable, 140cm x 200cm	10 & 50	
	LGF140200H	Slide, Flat Slide Sheet with Handles, Washable, 140cm x 200cm	10 & 50	
	LGT046060	Slide, Tubular Slide Sheet, Washable, 46cm x 60cm	10 & 50	
	LGT055120	Slide, Tubular Slide Sheet, Washable, 55cm x 120cm	10 & 50	
	LGT072072	Slide, Tubular Slide Sheet, Washable, 72cm x 72cm	10 & 50	
	LGT072120	Slide, Tubular Slide Sheet, Washable, 72cm x 120cm	10 & 50	
	LGT072150	Slide, Tubular Slide Sheet, Washable, 72cm x 150cm	10 & 50	
	LGT072200	Slide, Tubular Slide Sheet, Washable, 72cm x 200cm	10 & 50	
	LGT100120	Slide, Tubular Slide Sheet, Washable, 100cm x 120cm	10 & 50	
	LGT200070	Slide, Tubular Slide Sheet, Washable, 200cm x 70cm	10 & 50	

Disposable Slide Sheet Product Codes

	Product Code	Product Description	UOI	NHS SC Code
Slide Disposable	SGF090200R	Slide, Flat Slide Sheet - Disposable, Roll of 40, 90cm x 200cm	1	
	SGT075120R	Slide, Tubular Slide Sheet - Disposable, Roll of 48, 75cm x 120cm	1	
	SGT120120R	Slide, Tubular Slide Sheet - Disposable, Roll of 30, 120cm x 120cm	1	
	SGT180120R	Slide, Tubular Slide Sheet - Disposable, 180cm x 120cm	1	

Versal System Product Codes

	Product Code	Product Description	UOI	NHS SC Code
Versal System	B-Redi100200-W	Versal, Patient Positioning Device , Standard Washable, 100cm x 220cm	10 & 50	
	B-VERSAL-SPUH	Versal, Patient Positioning Device , Perforated Single Patient Use, 100cm x 220cm	10 & 50	
	B-VERSAL-WH	Versal, Patient Positioning Device , Perforated Washable, 100cm x 220cm	10 & 50	
	B-REDI100200-FT	Versal, Patient Positioning Device , Single Patient Use, 100cm x 220cm	10 & 50	

Proning Product Codes

	Product Code	Product Description	UOI	NHS SC Code
Prone Intoprone	IP085190PRN	Patient Turning Device, Single Patient Use, 85cm x 190cm	1 & 25	
	IP090195PRN	Patient Turning Device, Single Patient Use, 90cm x 190cm	1 & 25	
Orbit	B-Orbit	Prone Pack Into and Out of Prone plus EGT072072 Sheet	1 & 25	

Product Codes

Product Range

Transfer Sheet Product Codes

	Product Code	Product Description	UOI	NHS SC Code
Transfer Sheet	NWF110190H	Transfer, Patient Transfer Sheet, Single Patient Use, 110cm x 190cm	10 & 25	
	NWF85190H	Transfer, Patient Transfer Sheet with Handles, Single Patient Use, 85cm x 190cm	10 & 25	

Boards Product Codes


	Product Code	Product Description	UOI	NHS SC Code
Board Prime	BB-Prime	Prime, Seated Transfer Board, SWL 200kg, 62cm x 26cm	1	
Board Q2	Q2BOARD-Q2S	Q2, Seated Transfer Board, SWL 285kg, 75cm x 34cm	1	
Board Lateral Folding & Rigid	QP-059150-F	Transfer, Lateral Transfer Board, Folding, 59cm x 150cm	1	
	QP-059150-R	Transfer, Lateral Transfer Board, Rigid, 59cm x 150cm	1	
	QP-059180-F	Transfer, Lateral Transfer Board, Folding, 59cm x 180cm	1	
	QP-059180-R	Transfer, Lateral Transfer Board, Rigid, 59cm x 180cm	1	
	QP-074150-F	Transfer, Lateral Transfer Board, Folding, 74cm x 150cm	1	
	QP-074150-R	Transfer, Lateral Transfer Board, Rigid, 74cm x 150cm	1	
	QP-074180-F	Transfer, Lateral Transfer Board, Folding, 74cm x 180cm	1	
	QP-074180-R	Transfer, Lateral Transfer Board, Rigid, 74cm x 180cm	1	
Board Original	QBB01	Original, Seated Transfer Board, SWL 145kg, 73cm x 35cm	1	
Board Straight	QET01	Straight, Seated Transfer Board, SWL 175kg, 73cm x 22cm	1	
Board Rolaslide	QRS01	Transfer, Equipment Roll Board, 55cm x 177cm	1	

Patient Elevation Product Codes

	Product Code	Product Description	UOI	NHS SC Code
Slings Lock & Loop Sling	LOCO-090	Patient Specific Sling for use with Hoists High Back, with Loop Type Fixing, Small	10	
	LOCO-091	Patient Specific Sling for use with Hoists High Back, with Loop Type Fixing, Medium	10	
	LOCO-092	Patient Specific Sling for use with Hoists High Back, with Loop Type Fixing, Large	10	
	LOCO-195	Patient Specific Sling for use with Hoists High Back, with Loop Type Fixing, Extra Small	10	
	LOCO-196	Patient Specific Sling for use with Hoists Standard Back, with Clip Type Fixing, Small	10	
	LOCO-197	Patient Specific Sling for use with Hoists Standard Back, with Clip Type Fixing, Medium	10	
	LOCO-311	Standing Aids, Manual and Electric and Accessories Rotastand stand aid-SOLO	1	
	LOCO-887	Patient Specific Sling for use with Hoists Standard Back, with Loop Type Fixing, Small	10	
	LOCO-888	Patient Specific Sling for use with Hoists Standard Back, with Loop Type Fixing, Medium	10	

How to order?

 info@gbukhealthcare.com

 **+44 (0) 1757 288 587**
Lines are open Monday to Friday 8:30am - 5:30pm

 **NHS Supply Chain**

Air Assisted Devices Product Codes

	Product Code	Product Description	UOI	NHS SC Code
Air Reusable	AP-018N	Washable, Air Transfer Device, 18 inch Wide	1	
	AP-018SN	Washable, Sani Liner Cover for Air Transfer Device, 18 inch Wide	1	
	AP-028N	Washable, Air Transfer Device, 28 inch Wide	1	
	AP-028SN	Washable, Sani Liner Cover for Air Transfer Device, 28 inch Wide	1	
	AP-032N	Washable, Air Transfer Device, 32 inch Wide	1	
	AP-032SN	Washable, Sani Liner Cover for Air Transfer Device, 32 inch Wide	1	
	AP-039N	Washable, Air Transfer Device, 39 inch Wide	1	
	AP-039SN	Washable, Sani Liner Cover for Air Transfer Device, 39 inch Wide	1	
	AP-050N	Washable, Air Transfer Device, 50 inch Wide	1	
	AP-050SN	Washable, Sani Liner Cover for Air Transfer Device, 50 inch Wide	1	
AirPal Blower	AP-PA1200	Transfer Equipment Airpal Inflation Device	1	
AirPal Stand	AP-700-1	Transfer Equipment Airpal Stand Standard	1	
Air Single Patient Use	AP-034SPS	Single Patient Use, Air Assisted Transfer Mattress, 34 inch Wide	1	
	AP-039SPS	Single Patient Use, Air Transfer Device, 39 inch Wide	1	
	AP-050SPS	Single Patient Use, Air Transfer Device, 50 inch Wide	1	
Airpal	AP-700-1	Airpal, Stand Standard	1	
	AP-PA1200	Airpal, Inflation Device	1	

Care Management Systems & Accessories Product Codes

	Product Code	Product Description	UOI	NHS SC Code
Versal Accessories	B-MDBA001-G	Versal, Bed Adaptor, Single, For use with Versal sheet, 88cm x 35cm x 21cm	1	
	B-MDSC002-02-G	Versal, Bed Adaptor Clamps, Pair, For use with Versal sheet, 10cm x 6cm x 2.1cm	1	
Airseat	BAS1200	Airseat, Cushions Pressure Relieving Chairs, 45cm x 45cm	1	
Versal On-Bed System	B-VERSAL-SET	Versal, Bed Adaptor Set, Contents: 1 x Adaptor, 1 x Pair of Clamps, 2 x Clamp Straps, 2 x Adaptor Straps, 4 x Tethers, For use with Versal sheet	1	
	B-VERSAL-A	Versal, Adaptor Accessories Kit, Contents: 50 x Clamp straps, 50 x Adaptor straps, 50 x Tethers, For use with Versal sheet	1	
	B-VERSAL-A10	Versal, Adaptor Accessories Kit, 10 x Clamp straps, 1 x Roll of Adaptor straps, 50 x Tethers, For use with Versal sheet	1	
	B-INBEDREPO-W	Versal, Complete In-Bed Reposition System, Washable, Aid for reducing friction force	1	
	B-INBEDREPO-SPU	Versal, Complete In-Bed Reposition System, Single Patient Use, Aid for reducing friction force	1	
	B-REPO2B	Versal, Repositioning Sling	1	
Versal Plus	B-VERSAL-PLUS	Versal Plus, Advancing Care Pack, 1 x Versal sheet, 1 x in-bed sling, drawstring bag, Aid for in-bed patient movement	1	
One Way Glide	OWG-3843	Banana Glide - One Way, 38cm x 48cm	1	
Turning	QHDPT01	Banana Turntable, 40.5cm Diameter	1	
	EZ-410	Turntable SWL 120kg - Size Diameter 41cm	1	
	EZ-950	Patient Turning System 162cm x 37cm SWL 150kg	1	
Transfer Handle	Wypklene-RXH-S	Banana Transfer Handles, 15.5cm - 55cm	1	
	Wypklene-RXH-D	Banana Transfer Handles, 57.5cm - 45cm	1	
Handling Belt	CFHB01	Banana Belt, 56cm - 81cm	1	
	CFHB02	Banana Belt, 71cm - 115cm	1	
	CFHB03	Banana Belt, 101cm - 141cm	1	
	EZ-900	Handling Belt Padded Size 107cm x 15cm (Waistline 26"-35") SWL 150kg	1	
	EZ-910	Handling Belt Padded Size 128cm x 15.5cm SWL 150kg	1	

How to order?



info@gbukhealthcare.com



+44 (0) 1757 288 587

Lines are open Monday to Friday 8:30am - 5:30pm



NHS Supply Chain

Reducing Work-Related MSK

Bed incidents
reduced by

80%

**8.9
million**

Days

**Working
days lost**

in the UK due
to work related
MSK disorders

**13,842
lost**

Days

The average
Trust **lost
13,842 days
due to MSK
disorders**

(2015)

In the
workplace



17%

Suffer
lower
limb pain



38%

Suffer
back pain



45%

Suffer upper
limb or
neck pain

Lateral
transfer
incidents
reduced by

20%

Compliance
in usage
increased by

80%

**£200
million**

Sickness absence for MSK
cases is **costing over
£200 million**

Reducing The Cost Of Pressure Ulcers

**£1.4b
to £2.1b**

Pressure ulcers
cost to the NHS


29%

GBUK Banana products
have helped in the
**reduction in grade
2 pressure ulcers**



4%

**of total NHS
Expenditure**

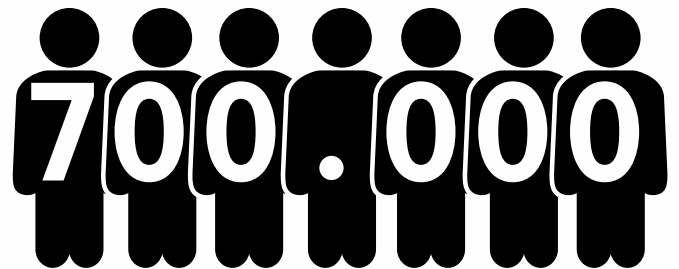


People nationally are
affected by pressure
ulcers each year

Notes

[illegible]



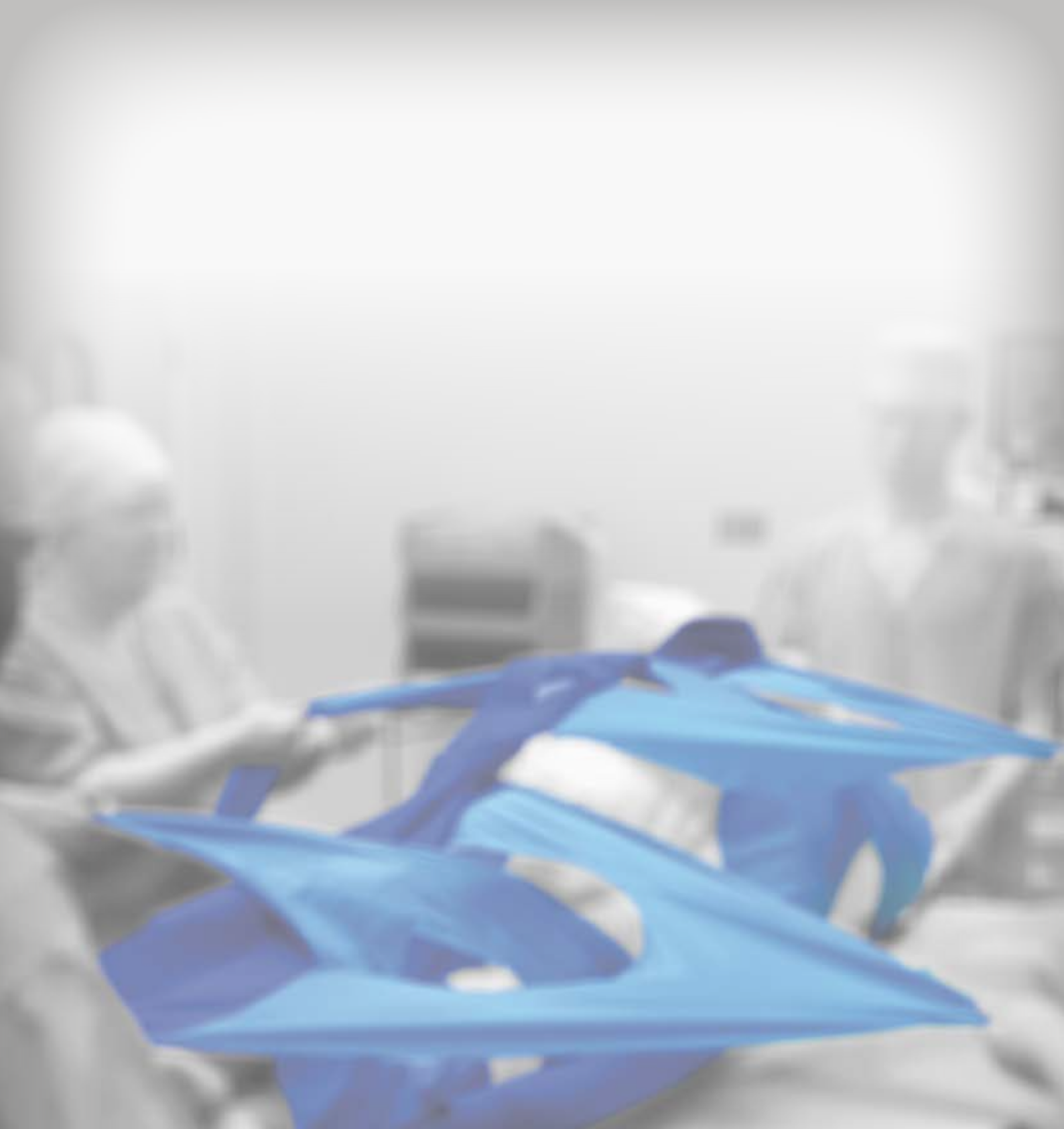

GBUK Group award-winning specialist companies are experts in the provision of enteral feeding, critical care and patient moving and handling products and technologies. GBUK Group companies are suppliers to every UK NHS Trust and to a rapidly expanding Export customer base.

www.gbukgroup.com



www.gbukbanana.com



GBUK Banana

Woodland House
Blackwood Hall Business Park
North Duffield, Selby
North Yorkshire YO8 5DD

T +44 (0) 1757 288 587
E info@gbukbanana.com